


# Instructions for use



## Protective Clothing

### put on

1. Step into legs of coverall and place coverall over safety boots.
2. Pull coverall towards arms and shoulders.
3. Pull zipper halfway up. Put on face masks (or respirator) and goggles. Pull hood over as head coverings. Cover sleeves over gloves.
4. Stretch to ensure coverall fits appropriately for ease of work movement and optimum protection
5. Be careful that no contaminate comes into contact with the wearer.
6. Contaminated garment should be disposed in accordance with local regulation.
7. The boot shoe covers should be on the pants.
8. Wear boot shoe cover so that the shoes do not touch the outside of the shoes. Then lift them up on your cloth.

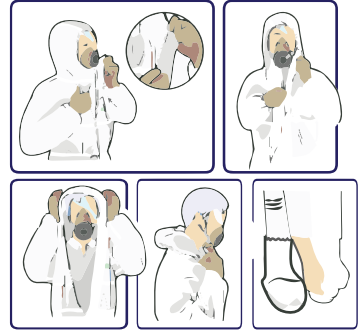

### Removing clothes

1. Release first arm from coverall and gloves. Release second arm and gloves from inside of coverall.
2. Sit and roll down contaminated coverall from inside towards outside.
3. Draw out legs from safety boots. Use clean hand to remove goggles. Fold or roll into a bundle and dispose coverall responsibly
4. This product is single use, do not reuse it.



The application and method of using this product is according to the label and IFU.

**COVER** each package contains

1 pcs coverall and 1 pair Boot shoe cover

ISO13688  
EN 14126



- Do not dry clean
- Do not iron
- Do not tumble dry
- Do not bleach
- Do not professional wet clean
- Do not wash
- protection against biological hazard
- Not Contains or presence of natural rubber latex
- Keep dry
- Do not re-use
- Keep away from sunlight

SIZE CHART Isolated clothing	SIZE	SIZE	SIZE	SIZE	SIZE
	S	M	L	XL	XXL
Shoulder width	148 CM	152 CM	152 CM	162 CM	164 CM
appropriate height	165-150 cm	175-160 cm	170-180 cm	180-200 cm	180-210 cm



Parque Empresarial  
do Padrão N° 11  
5460-343 Boticas  
Portugal  
+351 276 418287  
info@sterifast.com  
www.sterifast.com

Code: ST-MS-WI-09  
ver:01